



## VANGUARD PLATINUM

### PART NO. 10330

### PACKING LIST

- 1 - Vanguard Platinum Dust Collector (10330)
- 1 - Adapter 1½" Accessory x 2½" Hose End (VMC-1036)
- 1 - 2½" x 6' Hose (VMC-1023)

**IMPORTANT SERVICE INFORMATION:**  
 SAVE SHIPPING BOX AND FOAM FOR FUTURE SERVICE. SHIPPING WITHOUT PROPER FOAM MAY CAUSE DAMAGE TO UNIT

### ASSEMBLY INSTRUCTIONS for VANIMAN VANGUARD

1. Remove the unit and other parts from the box; check against the packing list.
2. Plug the power cord into a standard wall socket. Flip the black switch to "MANUAL." Try the unit by turning the switch to the "ON" position. The speed may be varied between low and high with the "SPEED" control slider. Speed selected depends on the job being performed and user preference.
3. The vacuum hose supplied slips onto the vacuum connection on top of the unit and the other end connects to the collection device desired. The ideal location for the unit is under the bench where the user works. The hose supplied is 2½" x 6' long. The hose cuff threads onto the hose with a left-hand thread. Longer hoses and adapters are available from your dealer. We recommend that you try to keep the total length of hose **less** than six feet between the unit and the dust collection point. Longer hose will reduce suction. See the accessories section for collection device details.

### SETTING UP THE AUTOPILOT

Introduction:

The **AutoPilot** allows you to control your dust collector from a micromotor or lathe. When you start your micromotor, the dust collector will start automatically. When you stop your micromotor, the dust collector will continue for a short period of time to allow for the residual dust to be cleared. **WARNING:**

The Autopilot will accommodate a micromotor, electric handpiece, or lathe that does not exceed **6 amps, 115VAC, total**

1. Flip the black switch to "AUTOMATIC", the dust collector should go off.
2. Plug one micromotor into the receptacle on the front of the Stone-Vac. Try the micromotor or trimmer and the dust collector should start automatically.

#### NSK Z500 Series Handpiece Users

#### From the NSK Z500 Users manual:

"Vacuum-coupled Mode

On some dental tables with vacuum dust collector, the motor may be used while being coupled with a dust collector. When such a dust collector is used, power consumption of the Z500 can be regulated so that the vacuum-coupled function can work. If you need coupling with a vacuum dust collector, select the mode as follows:



## VANGUARD PLATINUM

### PART NO. 10330

#### How to select the mode

While pressing Forward/Reverse Selector Switch (button), turn on the Power Switch, and the mode can be selected. A long beep indicates vacuum-coupled mode and 2 short beeps indicate non-coupled mode. Each time the switch selection is made, the mode changes between vacuum-coupled mode and non-coupled mode.”

---

### MAINTENANCE

---

1. **Filter Bag Replacement** - When a noticeable drop in suction occurs with the unit or it shuts down, it may be time to replace the filter bag. **DO NOT** attempt to clean out the filter bag. Replace with a new Vaniman filter bag. Change the filter bag by unscrewing the two thumb screws located on the side panel and remove the cover plate. The filter bag will now be exposed. Remove the used filter bag by sliding it off the inlet tube. The front of the cardboard reinforcement may need to be tilted downward slightly to remove bag easily. Once removed, check the inside of the filter bag compartment, if dust is present please clean the compartment. Cause for residual dust may be the filter bag was improperly installed. Vacuum out any dust before putting in a new bag. This will increase the life of the unit. Filter bag re-order number is **VMC-A400-5**, which can be ordered through your supplier.
2. **Installing a new filter bag** - slip the cardboard reinforcement, with rubber seal, over the inlet tube. You may need to tilt the front of the cardboard downward and slip the back edge up past the tube end, then rotate upward over the tube end. Carefully tuck the edges of the filter bag into the filter bag cavity. Replace the cover plate and gently tighten the thumb screws. Only gentle pressure needs to be used to make the gasket seal; the vacuum pressure will draw the cover tight.
3. **Circuit Breaker** - The unit is protected by a circuit breaker on the rear of the unit. In the unlikely event there is an overload condition the breaker may trip. If this happens reset the breaker by pushing on the white button until it clicks into place.
4. **Motor protection** - the Vanguard motor is thermally protected for against overload. If the unit becomes overheated, the thermal fuse will blow and must be replaced before the unit will operate. A Thermal Cutoff kit may be ordered through the factory at 1-800-VANIMAN (826-4626) or your supplier. **P/N 97013**
5. **Secondary filter** - the unit is equipped with a Post Filter which should be checked periodically and replaced when it is worn or soiled. See Accessories for the replacement part number and other filters available.

---

### RECOMMENDED USAGE

---

1. Individual work station (2½” hose) or two station (1½” hoses) dust collection for cutting, grinding and polishing of metal, porcelain, plastic and stone.
2. The unit may be used alone or in conjunction with the Large Vaniman Pre-Filter Accumulator. The Accumulator will trap 90-95% of the dust in the collection trap, extending the life of the Filter Bag by up to 10 times. **P/N VMC-A300**



## VANGUARD PLATINUM

### PART NO. 10330

3. The unit is **NOT RECOMMENDED** for use with aluminum oxide abrasive blasting. The fine abrasive particles will cause motor damage; such use will immediately void the warranty. Vaniman offers a SAND VAC, **P/N 10211** which is designed specifically for abrasive blasters. Contact your dealer for availability.

---

### ACCESSORIES & REPLACEMENT PARTS

---

The following items are offered to enhance the usage of your quality Vaniman unit. These items can be order through your supplier using the following part numbers. Please feel free to contact Vaniman Mfg. Co. with questions regarding our products.

| <i>Description</i>  | <i>P/N</i> |
|---------------------|------------|
| Odor Filter         | 96013      |
| Post Filter         | 96014      |
| HEPA Filter         | 96016      |
| Filter Bag (5 Pack) | VMC-A400-5 |
| Filter Bag (3 Pack) | VMC-A400-3 |
| Remote Switch       | VMC-A450   |
| Dual Remote Switch  | VMC-A455   |
| Foot Remote Switch  | VMC-A460   |