


---

## **PACKING LIST**

---

- 1 - Flex Hose
- 1 - Mounting Bracket
- 1 - Adapter 2-1/2" PVC to 2-1/2" Hose
- 4 - Mounting screws

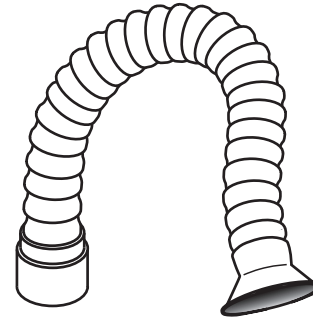

---

## **INTRODUCTION**

---

The Vaniman Flex Hose Kit is an easy way to provide a convenient and effective dust collecting station to the top or the front of your work surface. Use the mounting bracket attachment is made simple and easy. The Flex Hose Kit can be clamped onto the front of most work benches, if no hole is available. It can be positioned in many ways to collect dust, chips, mist, fumes and be pushed out of the way when not in use. Each link is snapped together and can be removed or added to added precision. Multiple nozzles are available as well as a slide gate valve.

---

## **ASSEMBLY INSTRUCTIONS**

---

Unpack the contents of the box and check the parts against the parts list and illustration above. Decide how you would like to connect the unit to your work area. There are two options:

1. **Tabletop Mount:** Remove the thumb screws from the Flex Hose, separate it from the Mounting Plate, and remove the Mount Adapter (Optional: Remove the metal Table Front Clamp from the Mounting Plate). Use the Mounting Plate as a guide and drill a 3.5" hole at the desired location. After the hole is drilled, insert the Mount Adapter into the Mounting Plate and place it in the desired location. Using the four Mounting screws, secure the Mounting Plate to the bench. Place the Flex Hose onto the Mount Adapter, align the holes on the side of the Flex Hose with the Mount Adapter, and secure together with the two thumb screws.
2. **Table Front Mount:** Loosen the screws on the Table Front Clamp and slide over the front of table. Tighten the screws to desired tightness.

Use the Adaptor to attach Vaniman 2.5" White Hose (VMC-1023).

---

## **ACCESSORIES ON NEXT PAGE---->**

---

**ACCESSORIES**

---

Vaniman offers multiple Flex Hose Nozzles as well as a Flex Valve:

Additional Ridged Hose -

3' section PN: 96001

1' section PN: 96002

Nozzles -

Round Nozzle PN: 96003

Rectangular Nozzle PN: 96004

Tapered Oval PN; 96005

Adapters -

3" PVC Drain Pipe (thin wall) PN: 96006

3" Sheet Metal Duct PN: 96007

Valves -

Flex Valve PN: 96062