


EXTRAORAL UPGRADE
FOR VANGUARD

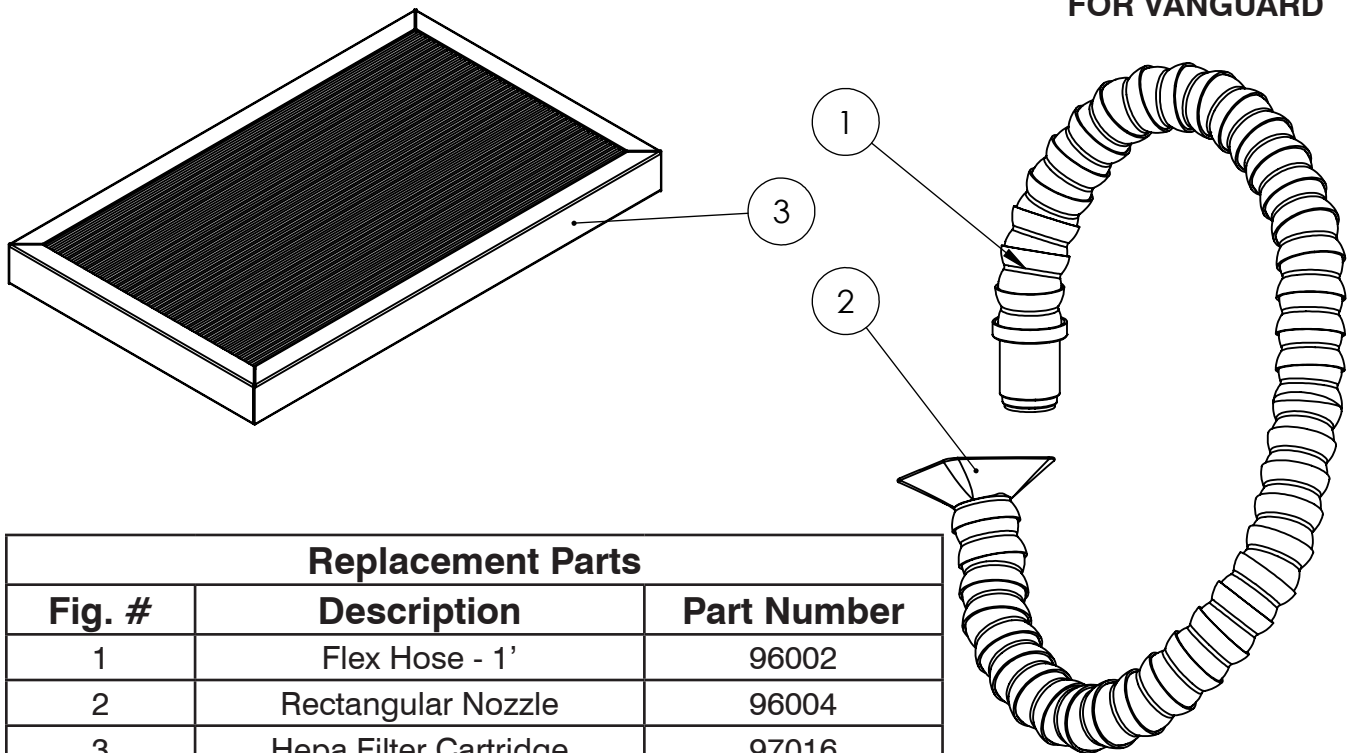

Replacement Parts		
Fig. #	Description	Part Number
1	Flex Hose - 1'	96002
2	Rectangular Nozzle	96004
3	Hepa Filter Cartridge	97016
Optional	Round Nozzle	96003
Optional	Tapered Oval Nozzle	96005

PACKING LIST

- 1 - 6' Flex Hose
- 1 - HEPA Filter Cartridge with Gasket

ASSEMBLY INSTRUCTIONS

1. Remove Flex Hose from packaging and connect it to the inlet tube on the top of the dust collector. Adjust the Flex Hose to desired position.
2. Remove the filter access panel (front cover) from the dust collector by unscrewing the two thumb nuts, and then either remove the filter bag from the unit or lift it up to expose the filter slot in the bottom of the unit.
3. Remove any packaging from the HEPA filter cartridge, and carefully align the frame opening over the filter slot of the dust collector.
4. Slide the filter into place with the gasket material facing upwards. Replace filter bag once HEPA filter cartridge is installed.
5. Replace the HEPA filter cartridge every 6-12 months or when the HEPA filter becomes excessively dirty and/or suction is impeded.