


ODOR FILTER & FRAME

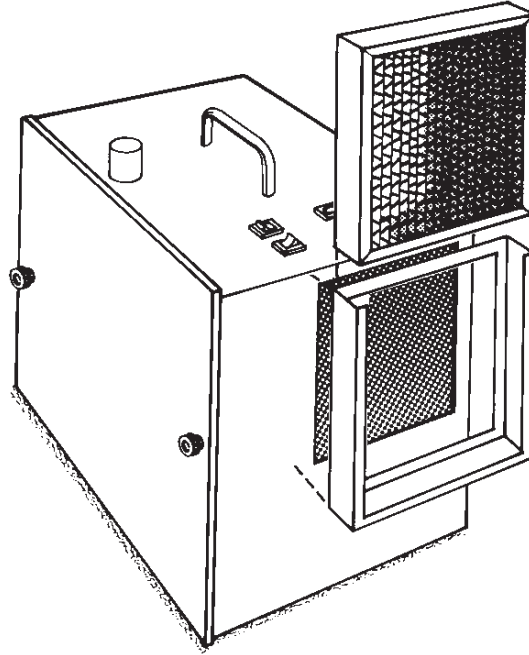

PACKING LIST

- 1 - Odor Filter (VMC-A410)
- 1 - Frame

ABOUT THE PRODUCT

This filter kit is designed to be mounted to the exhaust vent on Vaniman suction devices to filter fumes, odors, and vapors. It can be added to most Vaniman units or any other dust collector with an exhaust vent less than 6" x 7" in measurement.

INSTALLATION INSTRUCTIONS

1. Remove the odor filter from the frame by sliding it out of the open end of the frame.
2. Remove the protective paper from the adhesive strips on the frame.
3. Align the frame (as pictured above) with the opening over the exhaust vent of the unit, making sure the top edges are level with the top of the suction unit. Then, press firmly into place.
4. Remove the protective plastic from the odor filter and slide it into the filter frame.
5. Replace the filter as needed when odors begin to return or at least once per year.