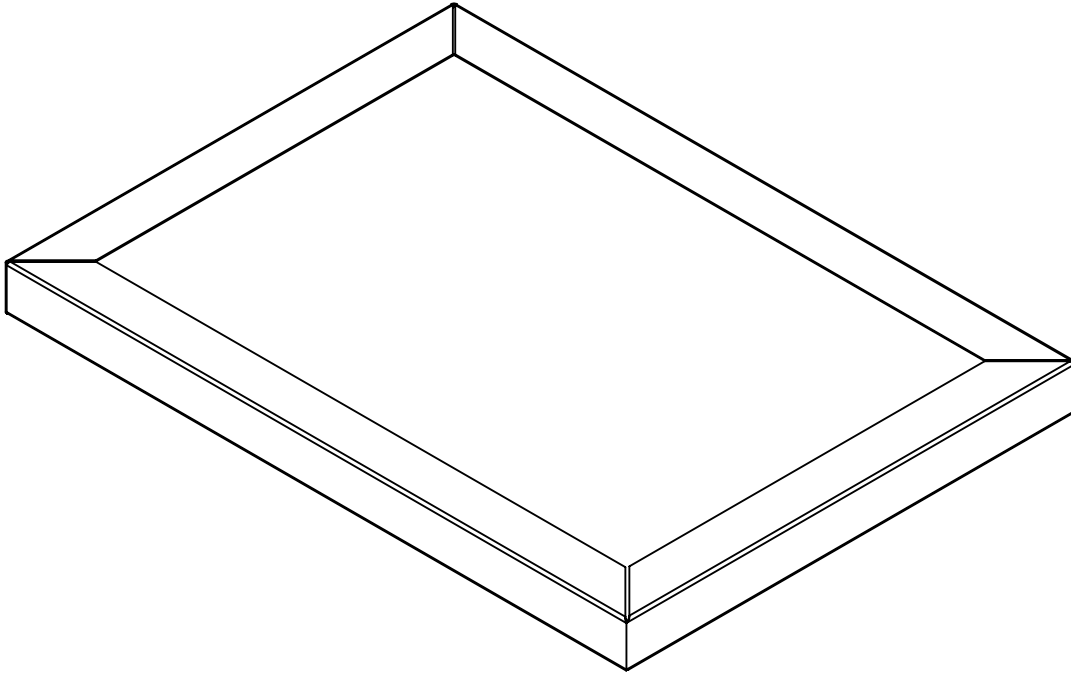

PRE-FILTER VANGUARD MOBILE 2.0



PACKING LIST

1 - Pre-Filter with Gasket

ABOUT THE PRODUCT

This filter is not intended to collect smaller sized airborne particles. This filter is specifically designed to help protect your HEPA filter media from larger airborne particles and moisture in order to maintain the filtering efficiency and ensure filter longevity.

FILTER REPLACEMENT INSTRUCTIONS

ATTENTION: To prevent additional spread of viruses or bacteria, it's advised to contact the CDC for proper handling protocol of specific pathogens.

1. Turn off the unit and remove the power cord.
2. Wearing full PPE (per the CDC's recommendations), set aside a sealable bag for disposing the filter(s).
3. Remove the filter access panel from the unit by unscrewing the four black thumbnuts on the front of the unit and set aside for later use. The Pre-Filter and HEPA filter should now be exposed.
4. Slide the pre-filter out and place filter into a sealable bag for proper disposal.



PRE-FILTER VANGUARD MOBILE 2.0

5. Remove all packaging from the replacement Pre-Filter. Carefully align the Pre-Filter frame with the outer walls of the unit, sit the filter onto the HEPA filter with the wire mesh facing down, and slide until the filter stops.
6. Remount the filter access panel back on the unit and secure with the four thumb screws.
7. Dispose of the sealed bag containing the used filter(s) per the CDCs recommendations.
8. Vaniman recommends replacing the Pre-Filter every three months and HEPA filter every six.

ASSOCIATED PRODUCTS

P/N 96097 - Vanguard Mobile 2.0 HEPA Filter with gasket

P/N 97045 - Vanguard Mobile 2.0 Filter Kit (2 Pre-Filters & 1 HEPA Filter)