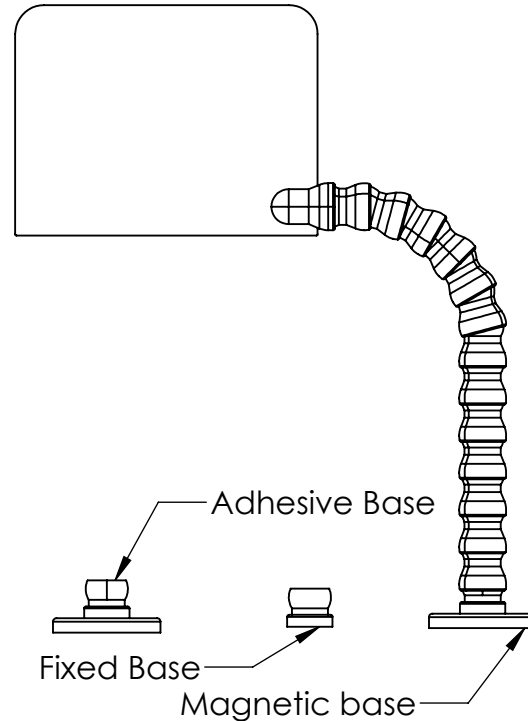

PACKING LIST

- 1 - FlexShield Assembly
- 1 - Fixed Base
- 1 - Magnetic Base
- 1 - Adhesive Base



INSTRUCTIONS

1. Remove the Flexshield Assembly, three base attachment options and all hardware from the packaging.
2. Select which of the base attachment options will work best for your application and attach the base in your chosen location by following the respective steps below:
 - a. **Magnetic Base:** Press the plastic end of the magnetic base onto the FlexShield assembly and then mount to any magnetic surface.
 - b. **Fixed Base:** At the location when you have chosen to mount the fixed base, drill a hole using a #7 drill bit (0.2" diameter hole) through the mounting surface. From the bottom of the mounting surface, feed the supplied 10-32 x 3/4" screw through the drilled hole and the fixed base as well, then secure it with the supplied 10-32 nut. Once the fixed base is secure, press the FlexShield assembly onto the base.
 - c. **Adhesive Base:** Remove the adhesive backing from the bottom of the adhesive base, then firmly press it onto the surface you have chosen to mount your Flexshield on to. Once the base is secure on the surface, press the Flexshield assembly onto the base.
3. Once your chosen base is secure to your mounting surface and you have attached the FlexShield assembly to the base, position the shield of the FlexShield over your work area so that it provides the most amount of protection from flying dust and debris.

REPLACEMENT PARTS

VMC-A391A - FlexShield Replacement Shield