


EXTRA RIGID CAPTURE HOSE

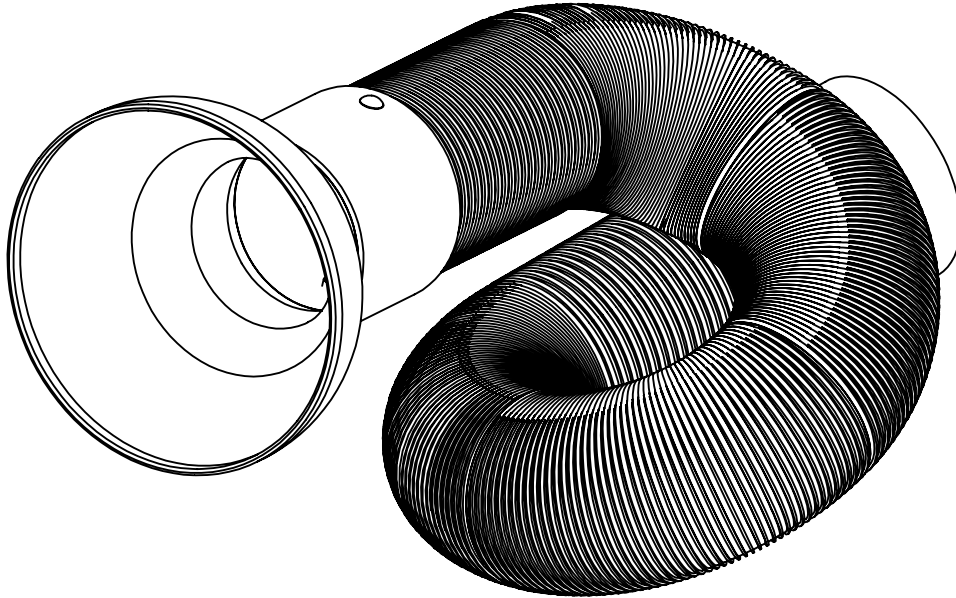

PACKING LIST

- 1 - Extra Rigid Capture Hose
- 1 - Nozzle

ASSEMBLY INSTRUCTIONS

1. Remove the hose and nozzle from all packaging.
2. Locate the end of the hose with the bayonet insert, which has a smaller inner diameter and contains internal grooves. Once found, take the nozzle and insert it into the bayonet. Ensure that the two pins on the nozzle slide into the grooves of the bayonet.
3. Once the nozzle has been inserted as far as it will go, hold the hose end and twist the nozzle in a clockwise motion. Twist the nozzle a full 90-degrees and you will feel a click, and the nozzle will be locked onto the hose.
4. Slide the other end of the hose onto the Vanguard Mobile 2.0 inlet tube, and adjust the hose to your desired orientation.
5. If your Rigid Capture Hose loses some of its rigidity and does not stay in the desired position, you can add rigidity back into the hose by performing the following steps:
 - a. Remove the hose from your unit and firmly grasp the hose with one hand about 1 foot below the nozzle end of the hose.
 - b. With your other hand, grasp the hose at the base of the black nozzle adapter and begin twisting the hose. As you begin to twist the hose you will notice that the hose diameter will either increase or decrease. If the hose diameter is decreasing, then it is becoming more rigid. If the hose diameter is increasing the hose is becoming less rigid and you need to reverse the twisting direction.
 - c. This process can be performed on any area of the hose to increase the rigidity and performance of your hose.



EXTRA RIGID CAPTURE HOSE

DISINFECTION INSTRUCTIONS

1. Only use a hot, soapy solution or Isopropyl Alcohol to disinfect your hose. Many of the chemicals used in typical disinfectants will cause the plastic to deteriorate quickly and break easily. Vaniman offers a surface cleaning spray for specifically disinfecting your hose safely and effectively. The Surface Disinfectant Spray can be ordered as **P/N 3556**.

NOTE: Never use any disinfectant or cleaning solution that contains **Hydrogen Peroxide**. If you are unsure if a particular solution is usable, please contact Vaniman for guidance.

2. The nozzle can be removed and washed with hot, soapy water, Isopropyl Alcohol, cold sterilized or Vaniman's Surface Disinfectant Spray. **THE NOZZLE IS NOT AUTOCLAVABLE.**

REORDER PART NUMBERS

PN: 96101 - Extra Rigid Capture Hose for the Vanguard Mobile 2.0

PN: 96104 - Extra Rigid Capture Hose for the Vanguard Gold 2X Mobile

PN: 96102 - Bulb Nozzle for Extra Rigid Capture Hose 5-pack