

FITTING INSTRUCTIONS



I. Cup the respirator in your hand with the nose piece at your fingertips allowing the headbands to hang freely below your hand;

II. Position the respirator under your chin with the nose piece up;

III. Pull the top strap over your head resting it high at the top back of your head. Pull the bottom strap over your head and position it around the neck

below the ears. Make sure straps are not twisted;

IV. Place the fingertips of two hands at the top of the nose piece. Mold the nose piece to the shape of your nose;

Note: Pinching the nose clip using one hand may result in a less effective respirator performance. Use two hands;

5. Cover the front of the respirator with both hands, sharply exhale and inhale, if air leaks around the nose, readjust the nose clip as described in Step 4, if air leaks at the respirator edges, work the straps back along the sides of the head to eliminate leakage. Repeat the procedure until the respirator fits properly;

Note: Do not wear with beards or other facial hair that may inhibit contact between the face and the edge of the respirator;

Warning: If you cannot achieve satisfactory fit, do not enter the contaminated area. See your Supervisor.

STORAGE AND MAINTENANCE

1. Propose to store these respirators in their original and undamaged packaging indoor with

Particulate Respirator-PT-N95F-01



relative humidity of less than 70% and temperatures of between 0°C~40°C;

2.The shelf life sees box bottom or inner packaging printing;

3.These respirators cannot be washed or microwave-heated.