


PACKING LIST

- 1 - Fishmouth
- 1 - Adapter (VMC-1006A)
- 1 - Flex Shield

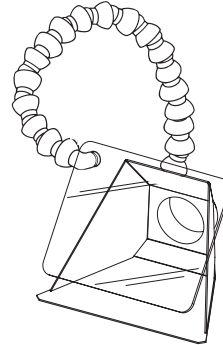

ASSEMBLY INSTRUCTIONS

1. Remove the Fishmouth and other parts from the packaging and check against the packing list.
2. Insert Adapter into the collecting port if needing to connect to 1.5" Grey Hose (see picture below).
3. Attach Flex Shield to the Fishmouth.

RECOMMENDED USAGE

1. The Fishmouth with Flex Shield is a work surface dust collector accessory that provides effective dust containment and removal. The Fishmouth is recommended for use with the Vanguard Gold 2X or Voyager dust collectors.
2. Position the Flex Shield in front of work to prevent debris from impairing vision during the grinding process.

ACCESSORIES & REPLACEMENT PARTS

- Shut-off Valve P/N: VMC-A380
- Replacement Flex Shield P/N: VMC-A391A

ADAPTER GUIDE

2 1/2" White Hose (VMC-1023)	1 1/2" Grey Hose (1450) w/ VMC-1009 end
<p>Fishmouth</p>	
<p>Shut Off Valve VMC-A380</p>	