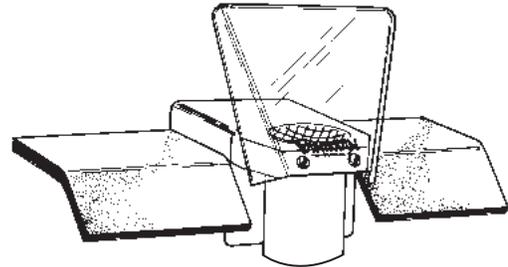

PACKING LIST

- 1 - Shield, Block & Handrest Assembly
- 4 - Wood Screws
- 4 - Double Stick Tape Strips
- 2 - Velcro Strips



INTRODUCTION

The Vaniman Jet-Port is an easy way to provide a convenient, effective dust collecting station at an existing table or work bench. Three options for attachment to the workbench are included for your convenience depending on the configuration of your work area. The oak work block is a convenient place to hold your work while grinding or polishing. The shield is strategically placed between you and your work to collect the dust produced without hampering your view or movement. Use of the Jet-Port will improve health and safety in your lab.

ASSEMBLY INSTRUCTIONS

Unpack the contents of the box and check the parts against the parts list and illustration above. Decide how you would like to connect the unit to your work area. There are three options:

1. Screw the unit to the bench with the two screws provided using the two holes in the front of the aluminum bracket and/or the two holes on top corners under the foam rubber.
2. Stick the unit to the bench top using the four strips of double stick tape provided. Place the four strips on the underside of the aluminum bracket, two near the table edge and two at the outer edge. Stick to a clean table surface.
3. Attach the unit to the bench top using the Velcro strips provided. Stick the "hook" side of the Velcro to the under-side outer edge of the aluminum bracket. Stick the "loop" side of the Velcro to the clean bench surface.

You may now choose how you want to connect the hose to the 2.5/3" Jet-Port. It may be connected directly to the bottom inside of the unit with a 2.5" hose (VMC-1023), or on the outside with a 3" hose (96020) and then held in place with a hose clamp (VMC-1039). To install with an elbow, use (VMC-1004). Be sure to support the hose so that it does not apply excess pressure to the Jet Port. For additional adapters and hose ends, please contact your Dealer or Vaniman Manufacturing Co.

REPLACEMENT PARTS

Shield Replacement: **VMC-A440A**

Dust Collector (Suggested): **10320 or 10201**