



## PACKING LIST

- 1 - Vanguard Gold 2X unit (10321)
- 1 - Pre-Filter Accumulator (VMC-A300)
- 1 - Power Cord
- 1 - 2 1/2" x 3' Hose (VMC-1023A)
- 1 - 2 1/2" x 6' Hose (VMC-1023)
- 1 - Adapter for 1 1/2" Accessory (VMC-1036)

**IMPORTANT SERVICE INFORMATION:**  
SAVE SHIPPING BOX AND FOAM FOR FUTURE SERVICE. SHIPPING WITHOUT PROPER FOAM MAY CAUSE DAMAGE TO UNIT

## ASSEMBLY INSTRUCTIONS

1. Mount the Accumulator according to the mounting instructions included with it, or assemble the bracket to the dust collector as illustrated. Remove the two screws provided on the side of the unit and use them to fasten the bracket to the vacuum.
2. Assemble the hose with the 1 1/2" hose end to the vacuum unit and the other end to the top of the Accumulator by firmly pushing the hose end into the adapter.
3. Connect the other hose from the side of the Accumulator to your station or accessory.