
PACKING LIST

- 1 - Selector Valve Tee
- 1 - 1.5" x 10' Hose
- 1 - Dust Brush
- 1 - Crevice Tool

KIT CONNECTION GUIDE

1. Remove all parts from its packaging and check against the packing list.
2. If connecting to your StoneVac:
 - A. Remove the hose connected to the back of the StoneVac, then slide the base end of the Selector Valve Tee directly into the inlet port of the StoneVac.
 - B. Reconnect your hose that was connected directly to the StoneVac to the empty side of the Selector Valve Tee (side without nipple).
 - C. Connect your 1.5" hose included in this kit to the nipple side of the Selector Valve Tee by sliding the hose end over the nipple until secure.
3. If connecting to your Accumulator:
 - A. Remove the hose connected to the inlet port of the Accumulator, then slide the base of the Selector Valve Tee into the inlet port of the Accumulator.
 - B. Reconnect your hose that was connected directly to the Accumulator's inlet port to the empty side of the Selector Valve Tee (side without nipple).
 - C. Connect your 1.5" hose included in this kit to the nipple side of the Selector Valve Tee by sliding the hose end over the nipple until secure.
4. To use the mill cleaning hose, turn the handle on the Selector Valve Tee so that there is suction through the 1.5" hose. Attach one of the hose end attachments to the free end of the hose and begin cleaning the inside of your mill.
5. Once cleaning is completed, return suction back to your mill by simply turning the handle on the Selector Valve Tee to the opposite side and suction will be provided through the 2.5" hose connected to your mill.

REPLACEMENT PARTS

- | | |
|--------------|------------------|
| Dust Brush | P/N: 1757 |
| Crevice Tool | P/N: 1758 |