


**PACKING LIST**

- 1 - Splash Guard
- 1 - Dust Pan
- 1 - Cap
- 1 - Cap with Screen
- 1 - Adapter (VMC-1006A)

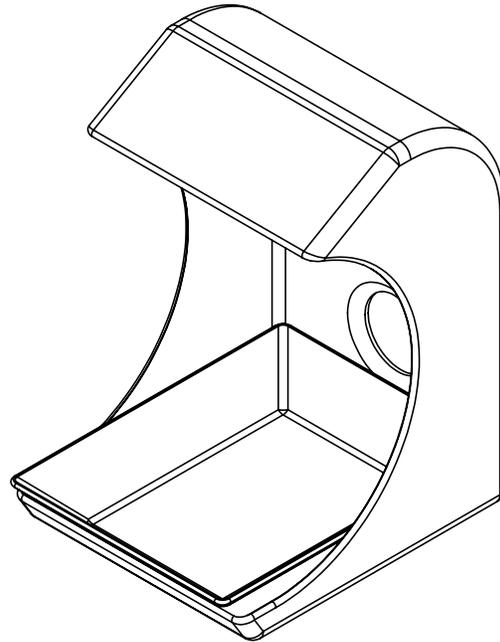

**ASSEMBLY INSTRUCTIONS**

1. Remove the Unit and other parts from the box and check against the packing list.
2. Attach a dust collector hose to the 2-1/2" port on the back. Use the supplied adapter to attach a 1-1/2" hose to the port.
3. Put cap with screen over the dust collector hose or adapter.

**RECOMMENDED USAGE**


1. The Splash Guard should be positioned appropriately to maximize dust and debris collection.

**CONNECTION GUIDE**


2-1/2" Hose w/ Manual Shut-off Valve



1-1/2" Hose w/ Manual Shut-off Valve

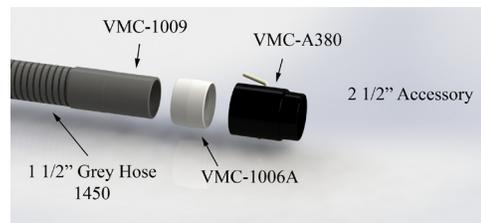

2-1/2" Hose to Splash Guard

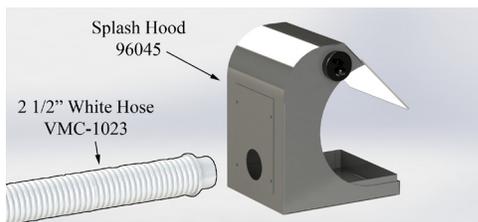

1-1/2" Hose to Splash Guard

