

**MODEL FIT GUIDE**

---

Some blasters may require trimming for proper shield fit. Some portions of the window may be exposed on some products.

**Cut to fit model guide:**

**97950** - Small - Sandstorm, Sandstorm Expert, Sandstorm Alpha, Sandstorm Dental, Sandstorm Professional, Problast II, Sandstorm Edge, VANiCAB

**97961** - Medium - Sandstorm2, Problast 2, and Cut & Clean

**97951** - Large - Sandstorm XL, Sandstorm Prestige, SandShell, Problast, Blast-It!, Problast3, Cut & Clean XL

---

**INSTALLATION INSTRUCTIONS**

---

1. Make sure bottom side of blaster window is clean. If glass is significantly etched, replace with a new window prior to installation of the Static Shield Protector.
2. Remove backing from Static Shield Protector.
3. Place one edge on blaster window and progressively lie shield on glass taking care to minimize bubbles and wrinkles. The correct side of the shield is the side that the Vaniman logo is readable from the front of the blaster.
4. **Tech Tip:** apply water or blue glass cleaner to window prior to installation.
5. Use a credit card or similar object to sweep bubbles out from under the shield protector. If you used water or glass cleaner for the application, use a credit card or similar object to sweep remaining moisture out from under the shield protector and repeat until all bubbles and excess moisture are gone. Let shield sit for 24 hours for the remaining liquid to evaporate for crystal clear results.
6. When surface is significantly etched, replace with new shield protector.