

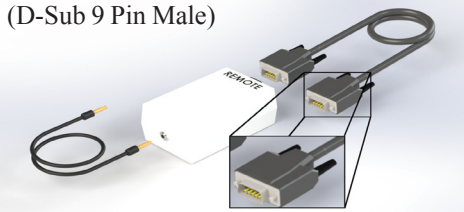
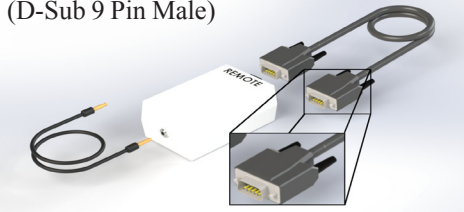
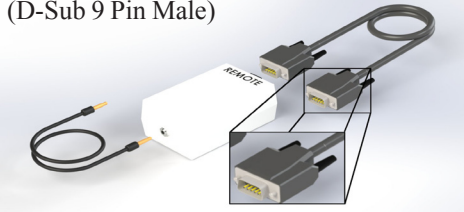
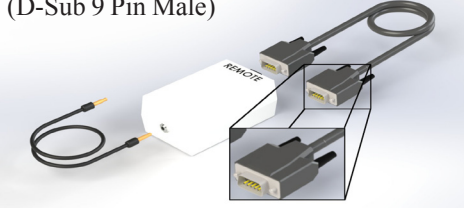

INTRODUCTION:

Vaniman's V6 SE and StoneVac Brushless SC models use a TRS Remote Port (3.5mm) for mill activation. Other units, such as the StoneVac II, use a Power Remote Port. This guide lists the mill-side remote options we offer and shows which Mill Remote is needed to connect your mill to our dust collectors. If you have any questions about which connection your setup needs, please contact us for assistance.

ASSEMBLY INSTRUCTIONS:

When connecting a Mill Remote to a dust collection unit, first power on the dust collector and verify it runs properly. After testing, turn the unit off, then connect the appropriate end of the Mill Remote to the dust collector's remote port and the other end to the mill. For detailed remote-operation instructions, refer to the user manual for your specific dust collector model.

REMOTE LIST

| Part Number | Mill Model | Remote Style | Render |
|-------------|--|--------------|--|
| 10535 | Ceramill | TRS (3.5mm) | (D-Sub 9 Pin Male)  |
| 10526 | Origin Proteus 5x | TRS (3.5mm) | (D-Sub 9 Pin Male)  |
| 10520 | Yenadent Dental M450i | TRS (3.5mm) | (D-Sub 9 Pin Male)  |
| 96077 | IMES 150i IMES 350i | TRS (3.5mm) | (D-Sub 9 Pin Male)  |
| 11072 | Roland DWX-50 Roland DWX-51D Roland DWX-52DC Roland DWX-52DCi Roland EGX-350 | TRS (3.5mm) | (TRS and MONO RCA)  |

| Part Number | Mill Model | Remote Style | Render |
|-------------|---|----------------|---|
| 96054 | LaserDental | TRS (3.5mm) | (3 Pole XLR Male)  |
| 96119 | Wieland Zenotec VHF Preciso M200(Jensen) VHF MasterMill VHF Bruxzir (Glidewell) VHF CAM 4-K4 VHF CAM 5-S1 VHF CAMV 5-S2 | TRS (3.5mm) | (4 Pin Din)  |
| 96078 | VHF K5 Impression VHF E5 | TRS (3.5mm) | (6 Pin Din)  |
| 96120 | XTCERA X-Mill 500 | TRS (3.5mm) | (GX16 4 Pin Female)  |
| 10525 | Digital Dental | TRS (3.5mm) | (.187" Quick Disconnect Male)  |
| 11063 | Dental Machine A5 Dental Machine C5 Dental Machine C6 | TRS (3.5mm) | (HAN 3A Female)  |

| Part Number | Mill Model | Remote Style | Render |
|-------------|---|-------------------|---|
| 11065 | ARUM 5X-300 | TRS (3.5mm) | (EU Power Cord)  |
| 10514 | Lava | TRS (3.5mm) | (US Power Cord)  |
| 10523 | Zirkon Zahn | TRS (3.5mm) | (3 Pin Power Remote)  |
| 96124 | Maxx Mill | Power Remote Port | (GX16 4 Pin Male)  |
| 96059 | Wieland Zenotec VHF Preciso M200(Jensen) VHF MasterMill VHF Bruxzir (Glidewell) VHF CAM 4-K4 VHF CAM 5-S1 VHF CAMV 5-S2 | Power Remote Port | (4 Pin Din)  |
| 11054 | Roland DWX-50 Roland DWX-51D Roland DWX-52DC Roland DWX-52DCi Roland EGX-350 | Power Remote Port | (TRS and MONO RCA)  |