


ODOR FILTER & FRAME

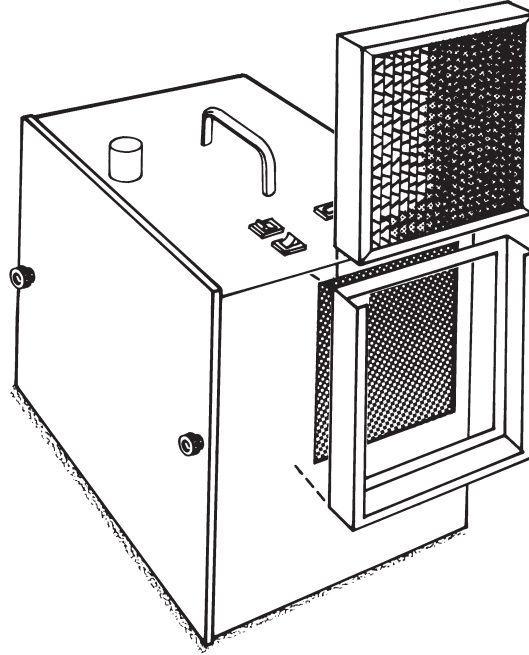

PACKING LIST

- 1 - Odor Filter (VMC-A410)
- 1 - Frame

ABOUT THE PRODUCT

This filter kit is designed to make your job more comfortable and safer by removing odors from acrylic Methylmethacrylate Monomer or most other odor-producing materials. The filter is designed to be added to Vaniman Dust Collectors or any other dust collector with an exhaust grill less than 6" by 7".

INSTALLATION INSTRUCTIONS

1. Remove the filter from the frame by sliding it out of the open end of the frame
2. Remove the protective paper from the pressure-sensitive adhesive strips on the back of the frame.
3. Carefully align the frame opening over the exhaust of the dust collector with the edges of the frame in line with the top and side of the vacuum unit. Press firmly into place.
4. Remove the protective plastic from the fume filter and slide it into the filter frame.
5. When the odors again become noticeable replace the filter cartridge.