

**MODEL FIT GUIDE**

---

Some blasters may require trimming for proper shield fit. Some portions of the window may be exposed on some products.

**Cut to fit model guide:**

**97950** - Small - Sandstorm, Sandstorm Expert, Sandstorm Alpha, Sandstorm Dental, Sandstorm Professional, Problast II

**97951** - Large - Sandstorm XL, Sandstorm Prestige, SandShell, Problast, Blast-It!

**97961** - Medium - Curved cabinets only - Sandstorm2 and Problast 2

---

**INSTALLATION INSTRUCTIONS**

---

Method A

1. Make sure bottom side of blaster window is clean. If glass is significantly etched, replace with a new window prior to installation of the Static Shield Protector.
2. Remove backing from Static Shield Protector.
3. Place one edge on blaster window and progressively lie shield on glass taking care to minimize bubbles and wrinkles. The correct side of the shield is the side that the Vaniman logo is readable from the front of the blaster.
4. Use a credit card or similar object to sweep bubbles out of protector.
5. When surface is significantly etched, replace with new shield protector.

Method B -- Recommended

1. Same steps as above except apply water or blue glass cleaner to window prior to installation.
2. Use credit card to sweep remaining moisture out of window -- repeat until all bubbles and excess moisture are gone.