


VANGUARD GOLD 2X

TWO STATION

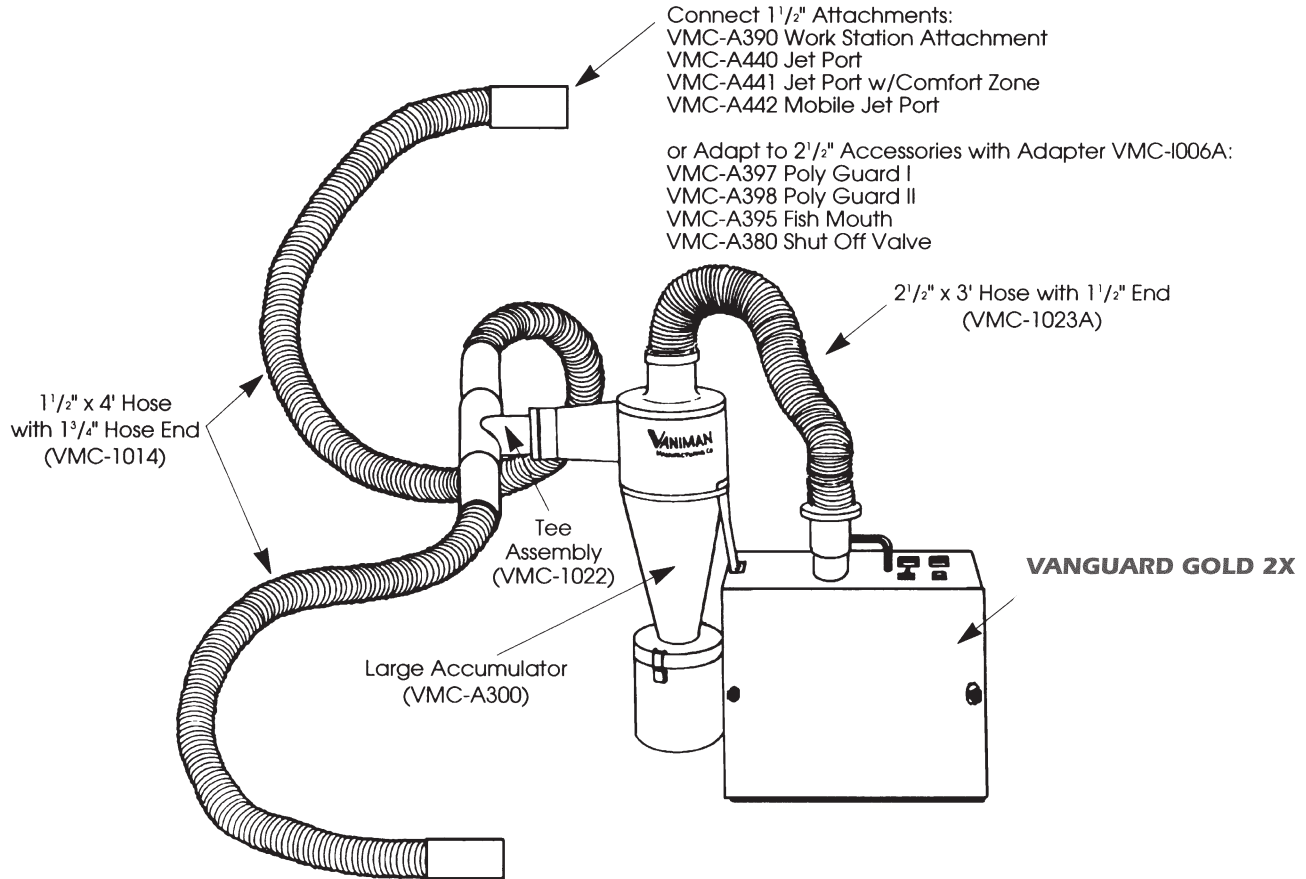

PACKING LIST

- 1 - Vanguard Gold 2X unit
- 1 - Pre-Filter Accumulator (VMC-A300)
- 1 - Power Cord
- 1 - Tee Assembly for Accumulator (VMC-1022)
- 1 - 2 1/2" x 3' Hose with 1 1/2" Hose End (VMC-1023A & 1008)
- 2 - 1 1/2" x 4' Hoses with 1 3/4" Hose Ends (VMC-1014)

IMPORTANT SERVICE INFORMATION:
 SAVE SHIPPING BOX AND FOAM FOR FUTURE SERVICE. SHIPPING WITHOUT PROPER FOAM MAY CAUSE DAMAGE TO UNIT

ASSEMBLY INSTRUCTIONS

1. Mount the Accumulator according to the mounting instructions included with it or assemble the bracket to the dust collector as illustrated. Remove the two screws provided on the side of the unit and use them to fasten the angle bracket to the vacuum.
2. Connect the 2 1/2" hose from the Voyager to the top of the Accumulator. Gently rotate and push the hose end into the black foam gasket. Moistening the foam with water will ease assembly. The hose may be shortened to minimize suction loss by cutting the plastic with a sharp knife and the support wire with wire cutters. The hose ends thread off with a left hand thread.
3. Assemble the 4' hoses to the Tee Assembly on the two tubes that are beveled (soften hose ends in warm tap water for easier assembly). Connect the Tee assembly to the side of the Accumulator (on the unbeveled tube already in the side of the Accumulator) and route the hoses to the collection devices. Shorten hose as required to minimize suction loss.