


VOYAGER 1 STATION

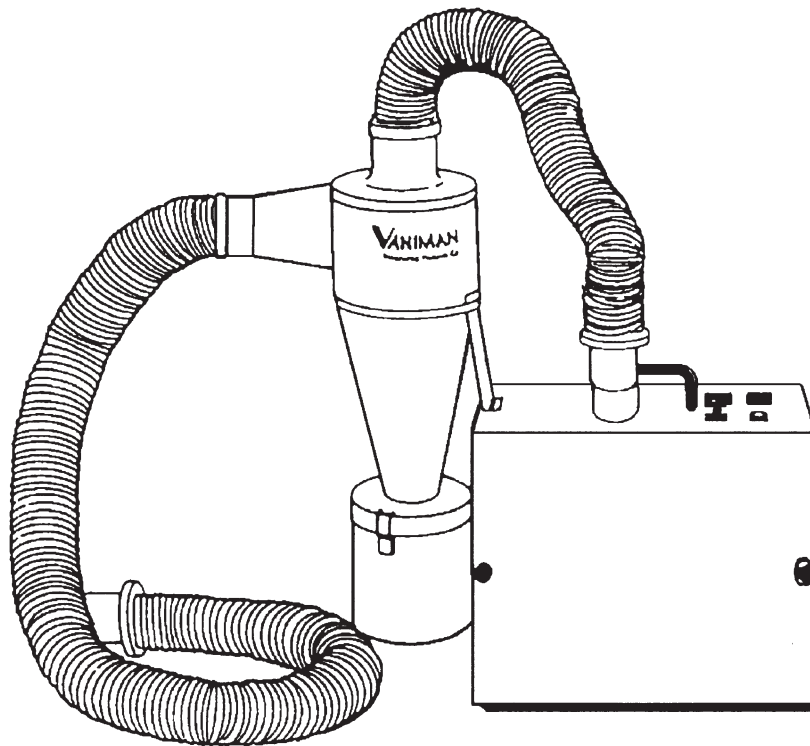

PACKING LIST

- 1 - Voyager (10630)
- 1 - Pre-Filter Accumulator (VMC-A300)
- 1 - Power Cord (1422)
- 1 - 2½" x 3' Hose (VMC-1023A)
- 1 - 2½" x 6' Hose (VMC-1023)
- 1 - Adapters for 1½" Accessory(VMC-1036)

IMPORTANT SERVICE INFORMATION:
SAVE SHIPPING BOX AND FOAM FOR FUTURE SERVICE. SHIPPING WITHOUT PROPER FOAM MAY CAUSE DAMAGE TO UNIT

ASSEMBLY INSTRUCTIONS

1. Mount the Accumulator according to the mounting instructions included with it or assemble the bracket to the dust collector as illustrated. Remove the two screws provided on the side of the unit and use them to fasten the bracket to the vacuum.
2. Assemble the hose with 1 1/2" hose end to the vacuum unit and the other end to the top of the Accumulator. Gently rotate and push the hose end into the black foam gasket. Moistening the foam with a small amount of water will help.
3. Connect the other hose from the side of the Accumulator to your station or accessory.